

PRODUCT	DESCRIPTION	PROS	CONS	ADDITIONAL INFO	ADA CODES
VALPLAST	Flexible denture base resin that is ideal for partial dentures	Blends with natural mouth tissue, comfortable and durable	Cannot be repaired and cannot add teeth to	Requires 5.5mm of occlusal clearance	D5225 - Maxillary partial denture D5226 - Mandibular partial denture
FRS	Thermoplastic injectable resin	Blends with natural mouth tissue, comfortable and durable	Cannot be repaired and cannot add teeth to	Requires 5.5mm of occlusal clearance	D5225 - Maxillary partial denture D5226 - Mandibular partial denture
DURAFLEX	Advanced thermoplastic material	Blends with natural mouth tissue, comfortable and durable, can be repaired and have teeth added to		Requires 5.5mm of occlusal clearance	D5225 - Maxillary partial denture D5226 - Mandibular partial denture
VITALLIUM 2000 CAST PARTIALS	Partial denture using a wide range of teeth and acrylic finishes	Smaller, lighter, offers good fit, adjustability and fracture resistance, biocompatible			D5213 - Maxillary partial denture - cast metal framework with resin denture bases <i>(including any conventional clasps, rests and teeth)</i> D5214 - Mandibular partial denture - cast metal framework with resin denture bases <i>(including any conventional clasps, rests and teeth)</i> D5281 - Removable unilateral partial denture - one piece cast metal <i>(including clasps and teeth)</i>
CU-SIL PARTIAL DENTURES	Partial with soft elastic gasket that clasps the neck of natural teeth	Prevents tooth loss, improves longevity of loose periodontally involved teeth	Cannot use on multiple teeth - patient will have trouble seating		D5211 - Maxillary partial denture - resin base D5212 - Mandibular partial denture - resin base
BRUX - EZE	Elasticized thermo-plastic mouth guard which provides comfort and durability	Cuspid guidance			D9940 - Occlusal guard, by report D9941 - Fabrication or athletic mouthguard
ECLIPSE HARD NIGHT GUARD	Clear resin night guard	Cuspid guidance and posterior disclusion		Cuspid guidance and posterior disclusion are standard unless otherwise specified	D9940 - Occlusal guard, by report D9941 - Fabrication or athletic mouthguard
ECLIPSE "HEAT & SEAT" GUARD	Clear resin night guard with an additional layer of soft material	Exceptional comfort and easy delivery, cuspid guidance	Must heat in warm water before seating	Softens when warmed, providing comfort and easy delivery; canine guidance/posterior disclusion are standard unless otherwise specified	
SOFT NIGHT GUARD	Constructed from soft thermoplastic	Customized version of a soft mouthguard	Not as durable as Eclipse		D9940 - Occlusal guard, by report D9941 - Fabrication or athletic mouthguard
COMFORT H/S HARD BITE SPLINT	Hard, clear, full coverage splint with even, flat contacts with opposing teeth	Occlusal thickness of the finished product is approximately 2.5mm	No canine guidance, posterior guidance or posterior disclusion		D9940 - Occlusal guard, by report D9941 - Fabrication or athletic mouthguard
COMFORT H/S HARD SOFT BITE SPLINT	Hard, clear, full coverage splint with even, flat contacts with opposing teeth, inner surface has 1mm layer of soft material for increased surface and ease of seating	Occlusal thickness of the finished product is approximately 2.5mm	No cuspid guidance or posterior disclusion		D9940 - Occlusal guard, by report D9941 - Fabrication or athletic mouthguard